

Broda Product Cleaning & Disinfecting Guide



Broda's products provide infection control by utilizing primarily vinyl upholstery. Cleaning and sterilizing Broda's products are easy for facilities and fundamental to minimizing healthcare bacterial and viral infections. The properties of vinyl upholstery intrinsically provide infection control measures:

- **Impermeable to moisture** – creates a fluid barrier.
- **Minimally textured** – ease of washing and sterilization.
- **Durable surface** – tears, scuffs and cracking resistant.

Upholstery	<i>Inclination, Interlace, Savoy, Kaleidoscope, Silververtex</i>	<i>Beeline</i>
Content	<i>Vinyl / Urethane Topcoat</i>	<i>100% Polyurethane/ PVC Free</i>
Finish	<i>Stain Resistant</i>	<i>N/A</i>
Performance	<i>100,000 D.R. UV Stabilized Pigments: Xenotest: AATCC Blue Wool / 1,000 hrs. No Heavy Metals Formaldehyde Free</i>	<i>500,000 D.R. Hydrolysis-Meets or exceeds ASTM D3690 02 Grade A requirements Reduced Environmental Impact: GREENGUARD Certified, PVC-Free</i>
Flame Resistance	<i>California T.B. 117-2013 NFPA 260 Class 1</i>	<i>California T.B. 117-2013 NFPA 260 Class 1</i>
Maintenance/Cleaning	<i>1:4 bleach / water, peroxide, & quaternary ammonium at the recommended dilution*</i>	<i>Water based Cleaner, 1:4 Mix of bleach & water</i>

For more information please visit the [Centers for Disease Control and Prevention \(CDC\) Guideline for Disinfection and Sterilization in Healthcare Facilities](#)

**Healthcare facilities maintain disinfecting regimens to reduce the spread of infections. There is a wide variety of available biocides that include bleaches, peroxides and quaternary ammonium products, among others. They should always be used at the recommended dilution, never in concentrated form. To prolong the life of vinyl or any other coated fabric, they should be rinsed off after the recommended residence time. It should be noted that misuse of disinfectants is the major source of surface deterioration.*